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(EN) Silver Breast Pads - Instructions for Use

Dear Customer,

Thank you for choosing our Silver breast pads! The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully - particularly the usage - before using this product for the first time. We will be glad to answer any questions and to receive any suggestions you might have!

Your ELANEE-Team

Please follow the instructions
and retain the instruction sheet and the packaging!

Made in Germany

1. Field of application

These instructions for use refer to the following product:

REF 218-00 Silver Breast Pads

2. Product content

- 2 Silver Breast Pads
- 1 Instructions for Use

3. Purpose and intended use

During breast feeding breast nipples may become sore and dry. The exsudate and the leaking milk are absorbed by the nursing pads to keep the breast dry. The silver nursing pads have a cool feeling to them when applied to the skin and permit air to circulate. They are soft and do not irritate the sensitive skin.

4. Functionality

The damp warm atmosphere within the nursing pads encourages the formation of bacteria that will then attack the breast. Thus the skin becomes irritated, the breasts may become inflamed and cause a mastitis. Also, encouraged by a warm temperature the bacteria may develop a disagreeable, rancid smell. Due to its antibacterial activity, silver (CAS-Nr. 7440-22-4), an active biocide agent, reduces colonization and repopulation of microorganisms in the breast pads and prevents odors. Wool and Lyocell are natural yarns that absorb and store liquid extremely well, which means that breasts and clothing remain dry.

5. Usage

Place the side of the nursing pads containing the silver yarn onto the breasts. You will recognise the correct side by its grey melange and the fine yarn thread. Use the nursing pads just as you would use any other standard pads between each breast feeding.

6. Cleaning and disposal instructions

Please soak in lukewarm water with a small amount of detergent, rinse well, absorb water with towels, pull into shape and dry flat. Adding a bit of vinegar to the water helps prevent matting. A certain degree of odor is normal for wool.

The Silver Breast Pads can be disposed of with the normal household waste.

Separate the individual parts of the packaging according to material and dispose of them in the correct recycling system. Observe the local and regional guidelines for disposal.

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